

# Millie Mova

Less effort, less risk, better care



If it looks easy,  
that's because it is



**B | E | S**  
HEALTHCARE

(+44) (0) 117 966 6761  
info@beshealthcare.net  
www.beshealthcare.net



# “The single best invention for safe manual handling of people since the hoist”

**Gavin Wright, Moving & Handling Consultant**

The Millie Mova is a unique, easy to use, and safe manual handling system designed to help a carer move an elderly or disabled person safely while seated in a chair. The Millie Mova attaches to the base of most four-legged chairs, and has a foot-controlled mechanism that allows the carer to engage the wheels to glide a chair short distances (such as into and away from a table) with an occupant in place, without risking possible injuries to their back, shoulder, knees, or hips while moving the person.

Unlike other solutions to the problem of moving users whilst safely seated, a chair fitted with the Millie Mova retains high stability, and incorporates an automatic brake, when unoccupied. This means that a person unsteady on their feet can lean on the chair without it sliding away from them, which otherwise could lead to a fall.



Scan for more info




and videos!

## Millie Mova Benefits

- Fits most dining chairs
- Simple to install
- Easy to use with minimal effort
- Reduces risk of injury to carer
- Improves user dignity and safety
- Extends chair and floor life
- Discreet, unobtrusive design
- Easy to clean for infection prevention
- Low maintenance

## How it works

Press down on the Millie Mova from behind to engage the wheels, push the occupant forward under the table, then release your foot to engage the brake automatically.

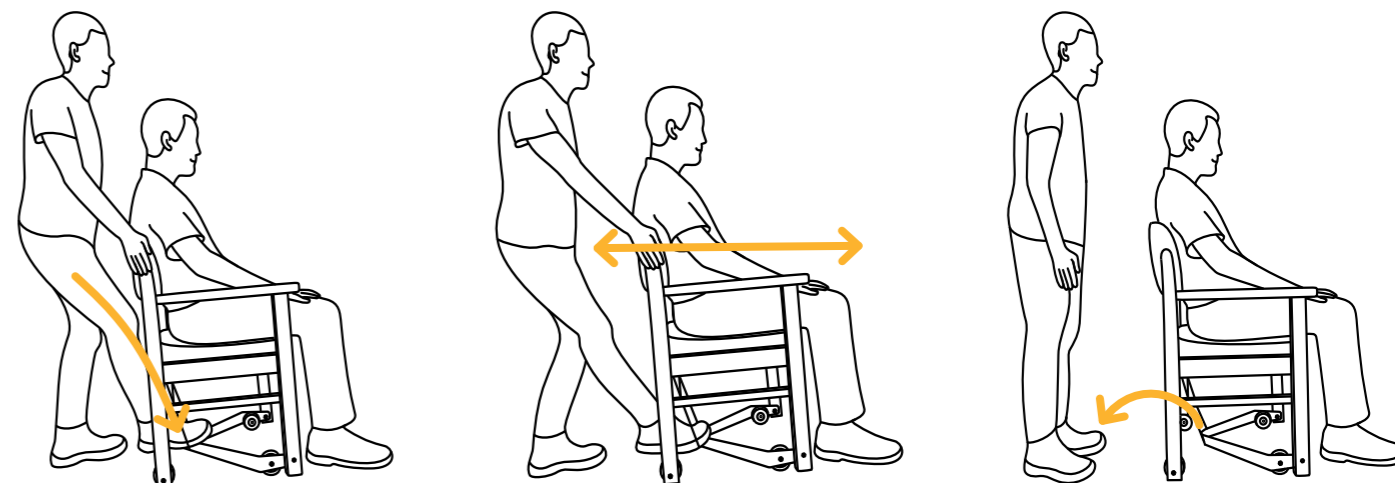

1

Press Millie Mova frame down to engage the wheels

2


Roll the chair forwards or backwards under a table

3


Release Millie Mova frame to engage the automatic brakes

## What you get


Choose your Millie Mova frame in either 40 cm or 47 cm width. Each kit includes a drill bit for easy installation. An optional pedal is available to extend the frame depth for larger armchairs, improving reach and ease of use.

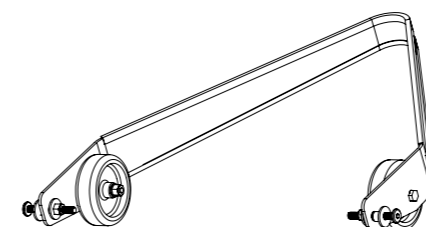

Millie Mova Frame (40 cm or 47 cm)

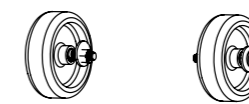

Rear Wheel Assemblies

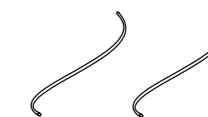

Bungee Cords



Rubber Feet

## Millie Mova Product Codes

Part Number	Description
Millie-Mova - 40	For chairs up to 40 cm wide
Millie-Mova - 47	For chairs up to 47 cm wide
Millie-Mova - 40M	For chairs with metal legs up to 40 cm wide
Millie-Mova - 47M	For chairs with metal legs up to 47 cm wide
Millie-Mova - 40PED*	For chairs up to 40 cm wide. Includes a pedal for easier access.
Millie-Mova - 47PED*	For chairs up to 47 cm wide. Includes a pedal for easier access.
Millie-Mova - 40PEDM*	For chairs with metal legs up to 40 cm wide. Includes a pedal for easier access.
Millie-Mova - 47PEDM*	For chairs with metal legs up to 47 cm width. Includes a pedal for easier access.



\*Millie Mova Pedal (optional extra)

# Millie Mova Testimonials

"We are delighted with the device – it has made such a difference."

**Liz, Dorset**

"In a care home or hospital where it could be fitted en-masse to a large number of chairs it makes perfect sense. It is well designed, easy to use and tough which is exactly what you want for those places."

**Review by Ergohacks.com**

"This gadget means that the carer does not harm themselves and the chair is not damaged either. Often the simplest ideas are the best and this is one of those occasions."

**Beverly Elliott, Abbeyfield Winchester**


"An invaluable aid to avoid staff injury. It also saves strain on our chairs."

**Mr P Darby, The Firs Residential Home, Lowdham**

## Also from BES Healthcare

### Penco Walker Glides

The Penco Universal Walker Glides from BES Healthcare improve manoeuvrability for wheeled walkers across varied surfaces, including carpet and concrete, for both indoor and outdoor use.

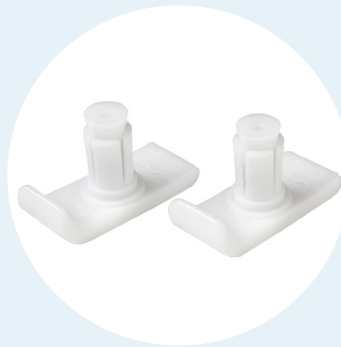

### Shear Comfort Sheepskin Footwear and Overlays

Our Shear Comfort range of footwear and overlays is great for those with vulnerable skin. The natural sheepskin helps regulate temperature, wick away moisture, reduce shear/friction, and redistributes pressure.



**B | E | S**  
HEALTHCARE

(+44) (0) 117 966 6761  
info@beshealthcare.net  
www.beshealthcare.net

## Supporting your needs

© 2026 BES Healthcare

EOE

BES00